

# Smartbox

## Lesson 1.7

### Creating jump cells

A jump cell allows a user to move between different pages in a grid set.

To create a jump cell

In **edit mode**, select a cell location and select **Create Cell**.

Select **Jumps** and then **OK**.

From the offered list (these will be existing grids within the current grid set), select the location for the **Jump**.


Click **OK** to create the jump.



**Tip:** use the keyboard shortcut **Ctrl + J** to make or edit a jump cell.

To jump to a new grid

After selecting **Jump**, select **New Grid**. Select the number of rows and columns needed give the grid a name and click **OK**. New grids made in this way will always contain a **Jump back** cell.

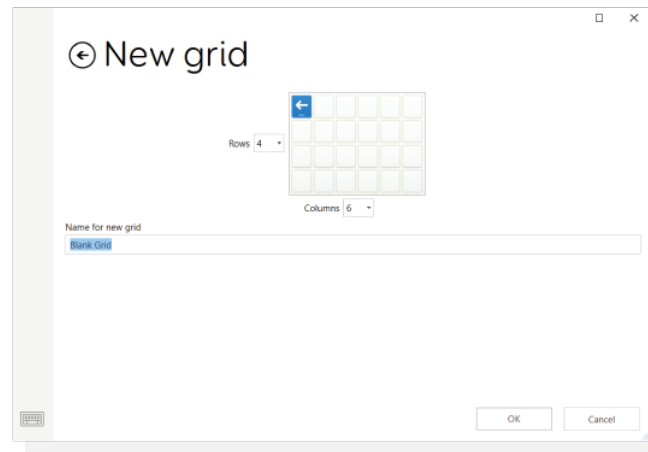


### Jump to a copy of an existing grid

To use an existing grid within the grid set as a template, select the grid from the list, select Copy Grid, give the grid a new name, click OK.



### Jump home or jump back

These two special jump cells will take the user to either their home grid, or the previous grid.

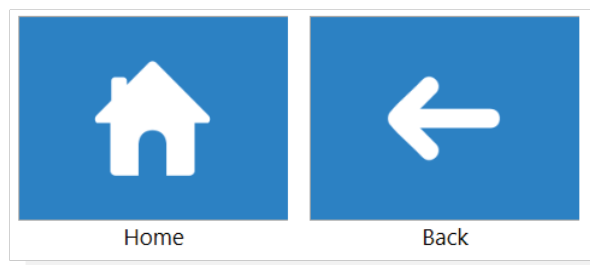

**Tip:** Use **Follow Jump** to navigate to your new grid without leaving **Edit mode**. You can also follow a jump with the keyboard shortcut **Ctrl + F**.

