

Tips for cleaning your device


Keeping your device clean is even more important now. Here are some handy tips to help you.

When do I need to clean my device?



1. When it's passed between different people.
2. When it's moved from one location to another (e.g. between home and school).
3. Whenever you pack or unpack your device.

How should I clean it?

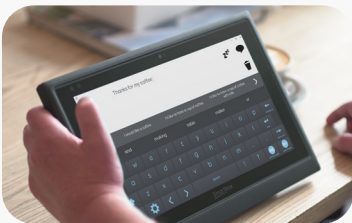

1. We recommend using Clinell Universal wipes or similar (containing at least 70% alcohol).
2. Other cleaning products can be used, just check they are effective against Coronavirus and are not harmful to the device.
3. When using other products we recommend testing a small area first to check that they are not harmful.

Things to remember



1. Do not allow liquids or aerosols to enter the ports or vents (see diagram).
2. Do not clean the device while it's charging.
3. Do not use overly strong cleaning products that could cause harm to the device or your skin.

Smartbox