

Grid Pad

Power Bank manual

Created by

Smartbox



Grid Pad Power Bank Manual (Version 2.0)
English (UK)

Copyright 2019 Smartbox Assistive Technology
thinksmartbox.com



Overview

Your Grid Pad Power Bank allows you to charge your device when away from a mains power supply.

What is included

- Power Bank unit
- Power Bank charging adapter
- Grid Pad 12 charging cable
- Grid Pad 8 and 10 USB charging cable
- Grid Pad Pro / Eye 11 charging cable
- Grid Pad Pro / Eye 13 charging cable
- Wheelchair carry bag

Supported devices

Grid Pad Power Bank supports the following devices:

- Grid Pad 12 (GP12A)
- Grid Pad 10 (GPP10A and GPG8A)
- Grid Pad 8 (GPP8A and GPG8A)
- Eye Gaze Tablet (EG13A)

- Grid Pad Go 8 (2016)
- Grid Pad Go 10 (2016)
- Grid Pad Pro 11 (2016)
- Grid Pad Pro 13 (2016)
- Grid Pad Eye 11 (2016)
- Grid Pad Eye 13 (2016)

1

Using your Power Bank


Your Grid Pad Power Bank has four power outputs. Plug the correct lead for your device into the output socket, then connect this to your device.

If you are charging a Grid Pad 12 the blue end of the charging cable connects to your device.

Press the **On / Off** button to begin charging.

Once your device has charged, disconnect it from the Power Bank and it will automatically shut down after 30 seconds.

If you leave the device plugged in to the Power Bank, it will continue to use the Power Bank's battery until it is depleted.

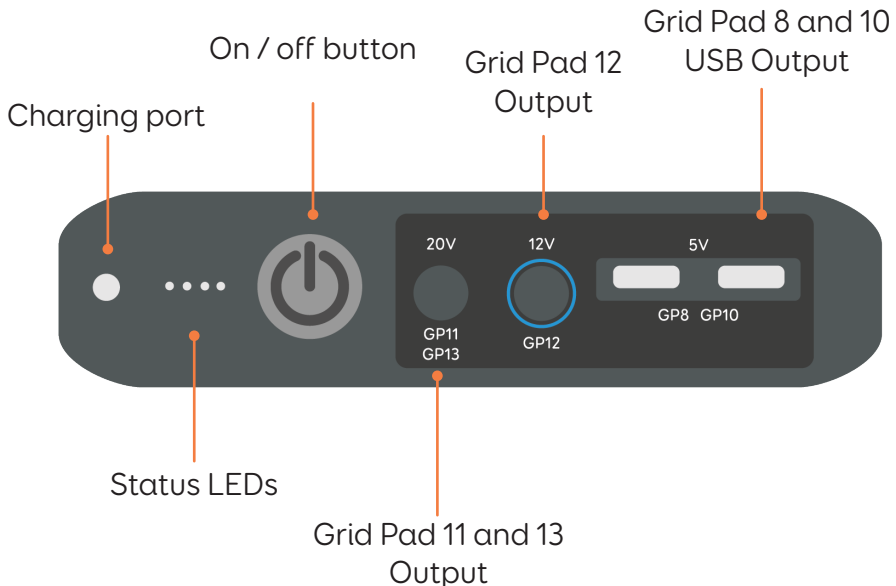

Charging your Power Bank



Attach the Power Bank charging cable to the charging port and connect the cable to a wall socket. To charge the Power Bank completely will take between 6 - 8 hours.

LED display

When in use, the status LEDs will display the remaining charge of the Power Bank.

0 - 25%	● ○ ○ ○
25% - 50%	● ● ○ ○
50% - 75%	● ● ● ○
75% - 100%	● ● ● ●
Low voltage alarm	● ○ ○ ○

When charging, the status LEDs will display the current charge of the Power Bank.

0 - 25%	● ○ ○ ○
25% - 50%	● ● ○ ○
50% - 75%	● ● ● ○
75% - 99%	● ● ● ●
100%	● ● ● ●



Additional guidance

Cycling the battery every six months is recommended to maintain the Power Banks battery. A cycle is the process of taking the battery from 0% charge to 100%.

Product information

Battery	Li-Polymer battery
Capacity	50000mAh
Charger input	AC 100v - 230v DC 16.8v / 2.5A
Output	USB ports 5v / 2.1A DC ports 20v / 3A 12v / 2.5A
Cycle life	≥1000 times
Working Temperature	0°C - 45°C
Storage temperature	-10°C - 40°C
Product size	207 x 136 x 33 mm

Safety information

4

- Make sure you are using the correct charging socket for your device. Wrong operation can cause damage to both your device and the Power Bank.
- Only charge the Power Bank with the supplied power unit.
- Do not use the Power Bank in damp places such as bathrooms or near water.
- Keep the Power Bank out of reach of children.
- Do not attach your device to the Power Bank input port.
- Do not expose the Power Bank to fire.
- Do not expose the Power Bank to hot environments over 60°C.
- Dropping, shaking or applying weight to the Power Bank can damage the internal circuit and can cause damage to both the Power Bank and your device.
- Do not attempt to open or repair the product.
- The Power Bank is designed and tested with Grid Pad devices. Smartbox does not support using the Power Bank to charge other devices.
- Tilt warning - if carrying your Power Bank in the bag provided and mounting on a wheelchair, ensure this does not unbalance your wheelchair
- There are strict regulations for lithium ion batteries on airplanes. Different airlines have differing rules, so it is recommended to contact your airline before you travel.



Li-ion



Dispose of according to local regulations.



Power Bank manual

Smartbox

thinksmartbox.com

GPPB-MAN-B